



What Can I Find Out Using the Realize Language Site?


How are words used over time?

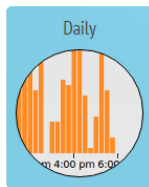

The **Daily** widget shows you how often a client uses words in 15-minute intervals. See what time of day is the best from a communication perspective.

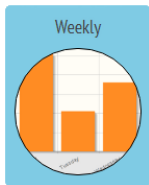


The **Weekly** widget shows how many minutes per day a client is using their device over a one-week period. See which days are more productive than others.

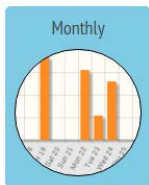

The **Monthly** widget measures minutes of device use over a complete month. See if there have been any changes over time.

Which words are being used?

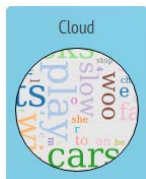

Create a fun, colorful word **Cloud** based on all the words a client has used in a time period of your choice. The size of the words is directly related to the frequency of use. A great way for a client to see what he or she is capable of doing.



The **Top Ten** widget creates a bar chart to show the ten most frequently used words. You can change the time period to show anything from all the words ever used to just a single day.




For a list of all the words a client has used, the **A – Z** widget provides an instant list. As with other widgets, you can set the time period in which the list will be created. Use this to compare the words used in one week versus another.



If you want a list of all the words a client has used and how frequently they've occurred, use the **List** widget. You can even copy and paste this list into a document or spreadsheet.

Which words have been spelled and which are pre-stored?

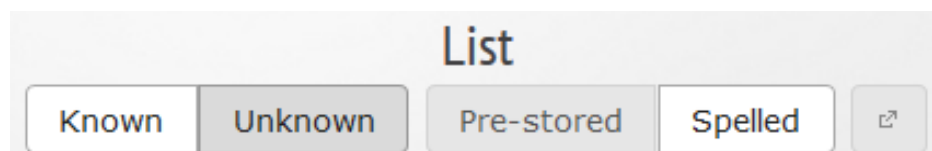


Using the **List** widget you can toggle between seeing all the words that have been **SPELLED** out and all those that are **PRE-STORED**. You can also create a list of all **SPELLED** words that are not real words – a really useful feature for monitoring and teaching literacy development.


What types of words are being used and how do they relate to Goals?



The **Parts of Speech** widget breaks down all your client's words into categories such as nouns, verbs, adjectives, pronouns, and so on. You can use this tool to isolate specific groups of words and determine target vocabulary.

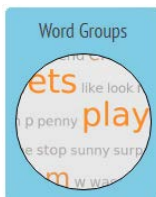

The Realize Language site has a target dictionary of over 100,000 words that your client can aim for! But it has a starter goal of 300 of the most commonly used words. The **Goals** widget shows how a client is doing as compared to this list. You can then track progress in relation to the 300-word goal.

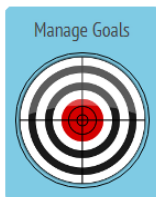

The Realize Language site includes a default set of 300 words that you can track. But you can create any list you want and add it to the site. So if you want to track progress against a specific set of 25 words, the **Manage** widget lets you enter and set then as the comparison list. You can even switch between vocabulary lists using a simple drop-down menu.

What sort of things are being said? And when?



Betty Beta » Log Fri Apr 3, 2015 - Tue Oct 13, 2015

Find Words... Hide Details 30

Week **Month** Hotspot Threshold: 97

	Sun	Mon	Tue	Wed	Thu	Fri	Sat
1 - 4							
5 - 11							
12 - 18							
19 - 25							
26 - 30							

Wednesday, April 8th, 2015

- I15
- H6 [TIME]
- E3 [VERB+S] takes
- J6 [UMBRELLA]
- I3 [TO+VERB] to need
- L7 [LOVE]
- J3 [ADJ] new
- F7 [DOG]
- F3 [VERB+ING] coming
- L7 [LOVE]
- L3 [ADJ+EST] newest
- I6 [YORK]
- G3 [VERB+ED] welcomed
- O7 [THMBS DN]
- D6 [UGLY]
- J3 [ADJ] ugly

- ① Use the **Find Words...** box to look for specific times when a word was used. A small black dot appears on the calendar in every ¼-hour period it was used.
- ② Click on any ¼-hour slot in the calendar to see everything that was said presented in the **Log Column** on the right.
- ③ Use the **Week** and **Month** tabs to switch between time periods. Use the arrow keys to go backwards and forwards either week-by-week or month-by-month.
- ④ Scroll up and down in the **Log Column** to see what was said, when it was said, and how it was produced i.e. was it a pre-stored word, a pre-stored sentence, or even a spelled word?
- ⑤ Change the level of detail you see in the log column. You can go from just seeing the words with a line across at regular intervals to seeing time, location, and type of coding being used. You choose.
- ⑥ The log column inserts an orange line every 30 seconds to make it easier to see when words are being used to create sentences. You can change this setting from 5 seconds to 120 seconds.
- ⑦ Use the drop-down **Date Range** button to change the calendar. If it's January 2014 and you want to see January 2011, just click in the **Date Range** and reset the time.